Baxter

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URGENT DEVICE
CORRECTION

11th of May, 2018

From: AO Company "Baxter"
125171, Leningradskoye Shosse 16a, build.1
Moscow, Russian Federation

To: Ministry of Health of Republic of Armenia

№ QA - 026 dd. 11.05.2018 r.

Subject: Field Safety Notice – Device correction

Prismaflex Control Unit - Combination Alarm - Voltage out of Range and Defective Blood Pump

Product names: Prismaflex 7.XX Row,

Product codes: 114870

Serial Numbers: All

Dear Sir/Madam,

Baxter Healthcare will be installing new firmware on all Prismaflex control units to address the potential for a small number of these units to exhibit a failure mode with the pump module electronics. The failure mode may result in a "Voltage Out of Range" malfunction alarm, which causes the device to enter a "safe state" and become inoperable until it is serviced. Baxter will be releasing new firmware that will prevent the malfunction from occurring.

The "Voltage Out of Range" malfunction alarm causes the Prismaflex control unit to enter a "safe state" by stopping all pumps and closing the return line clamp. This failure mode can occur at any time during use and may result in an interruption and/or delay in therapy. Patient harm is not expected as the blood may be manually returned to the patient. There have been no reports of serious injury associated with this issue.

A local Baxter service representative will contact the customers to determine the correction plan and schedule the firmware upgrade.

Our records indicate that 1 (one) customers have received this product in Armenia. For your information, please find attached the communication that is being sent to those customers.

FA-2018-014

Baxter is a registered trademark of Baxter International, Inc.

Page 1 of 5

Baxter

Should you have any questions, please contact Nataliya Popova by phone + 7 (495) 647 68 07 or by e-mail natalia_popova@baxter.com.

Yours Sincerely,

Nataliya Popova

Quality Head, Russia and CIS
AO Company Baxter

« 11 » May 2018

Addendum 1 – Distributor communication on 3 (three) pages.

May, 2018

Dear

Problem Description

Baxter Healthcare will be installing new firmware on all Prismaflex control units to address the potential for a small number of these units to exhibit a failure mode with the pump module electronics. The failure mode may result in a "Voltage Out of Range" malfunction alarm, which causes the device to enter a "safe state" and become inoperable until it is serviced. Baxter will be releasing new firmware that will prevent the malfunction from occurring.

Affected Product

Product Code	Product family	Serial Numbers	Distribution Dates
114870	Prismaflex 7.XX Row	All	2016

Hazard Involved

The "Voltage Out of Range" malfunction alarm causes the Prismaflex control unit to enter a "safe state" by stopping all pumps and closing the return line clamp. This failure mode can occur at any time during use and may result in an interruption and/or delay in therapy. Patient harm is not expected as the blood may be manually returned to the patient. There have been no reports of serious injury associated with this issue.

Attachment: Customer Reply Form
URGENT DEVICE CORRECTION LETTER DATED MAY 2018

Product Family: Prismaflex

Product names: Prismaflex 7,XX Row,

Product codes: 114870

Please complete and return one copy of this form per facility either by fax +7 (495) 647 68 08. or by e-mail russia_quality@baxter.com as confirmation that you have received this notification. A fax cover sheet is not required.

Customer Confirmation

We confirm that that we have have received the above mentioned letter, understood its content and have disseminated this information to our staff, other services and facilities.

We confirm that we have received the above mentioned letter, understood its content and have disseminated this information to our Customers

Facility Name and Address: <i>(Please Print)</i>	
Product code and Serial Number of Machine	
Reply Confirmation Completed By: <i>(Please Print Name)</i>	
Title: <i>(Please Print)</i>	
Email and/or Telephone Number <i>(Including Area Code):</i>	
Signature/Date: REQUIRED FIELD	_____ / ____ / ____

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GLOBAL DISTRIBUTION ALLIANCE
Business Partner

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